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Please read these instructions for use carefully - it contains important information on how to use this product. If you have any questions, please ask your pharmacist, your doctor, your eye care professional or contact us directly.

BAUSCH + LOMB

Bio
true

eye drops

Inspired by the nature of your eyes

0.24% sodium hyaluronate, sodium chloride, potassium chloride, disodium phosphate dodecahydrate, sodium dihydrogen phosphate-dihydrate and sterile water.

Intended use: eye lubricant and wetting and rewetting of both soft and rigid contact lenses during wearing, while also providing instant hydration for all dry eye symptoms.

What is Biotrue® rewetting drops? Biotrue® rewetting drops is a clear eye drop solution that contains 0.24% hyaluronic acid (as sodium hyaluronate). Biotrue® rewetting drops is very well tolerated by users due to its preservative free formulation. The eye drops are presented in an innovative preservative free multi-dose system (PFMD), which is easier to use. Once opened it can be used for 6 months.

What is the application of Biotrue® rewetting drops? Biotrue® rewetting drops is intended to instantly soothe and hydrate dry eyes. It naturally refreshes and calms your dry eyes relieving symptoms of tired, stressed and even teary eyes. Biotrue® rewetting drops is suitable for the wetting and rewetting of both soft and rigid contact lenses during wearing and improves the comfort of your contact lenses noticeably, all day long.

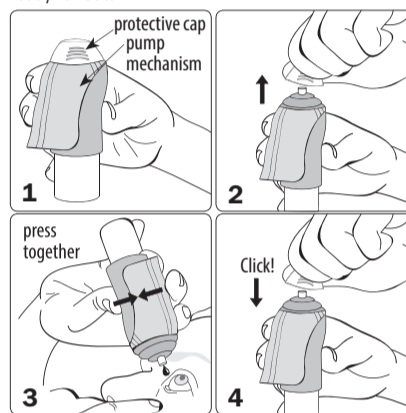
When you should use Biotrue® rewetting drops? Biotrue® rewetting drops is designed to remain on the surface of the eye providing instant hydration for all dry eye symptoms. It is an ideal product for everybody suffering from a range of symptoms of dry eyes such as tired, stressed, burning and/or teary eyes.

Dry eyes are caused by:

- external factors, such as extensive computer use, air conditioning, heating, cigarettes smoke, windy outdoor environments as well as wearing contact lenses.
- other factors, like certain drugs (such as the contraceptive pill, antihistamines, betablockers, antispasmodics and diuretics) or certain diseases that play a role in the development of ocular dryness.

How often should you use Biotrue® rewetting drops? Instill one drop of Biotrue® rewetting drops into the eye as the need arises. You can use it every day and as often as you want. There is no limit of duration of use.

How to use Biotrue® rewetting drops? Attention! Before first use and if the bottle has not been used for a while: Remove the cap and point the bottle downwards, press repeatedly approximately 20 times until a drop is released. Now the bottle is ready for use.

1. Wash your hands before using the product and hold the bottle as indicated.

2. Remove the protective cap from the tip of the bottle with a rotating motion before each use.

Avoid contact of the tip of the bottle with your fingers or any other surfaces to prevent any possible contamination of the solution.

3. Lean your head back slightly. Gently pull your lower eye-lid down with a finger of your free hand, hold the bottle in a vertical position over your eyes and press the pump mechanism together to apply one drop into the eye.

Avoid touching the surface of your eye with the bottle tip to prevent any possible injury.

Close your eye afterwards and slowly roll it in order to optimally spread the drop on the surface of your eye.

4. Immediately after use, shake-off any possible residual drops and replace the protective cap on the dry tip of the bottle.

When you should not use Biotrue® rewetting drops?

Do not use this product if you are hypersensitive to any of the ingredients.

Important information about using Biotrue® rewetting drops?

- Local hypersensitivity reactions have been very rarely reported. Treatment should discontinue in such cases.
- Safety information for patients with severe damage of the cornea: In very rare cases deposits of calcium phosphate on the cornea were reported during treatment with phosphate-containing products for ophthalmic use.
- Do not touch the eye surface with the tip of the bottle to prevent any possible injury of the eye surface or contamination of the solution.
- Shake-off any possible residual drops after application.
- Wait approximately 15 minutes before using another eye product.
- Do not use after the use-by date.
- Do not use if the package or bottle is damaged.
- Discard the medical device after 6 months once opened.

How to store Biotrue® rewetting drops?

Store at a temperature of +1°C to +25°C.

The design of the bottle allows you to use Biotrue® rewetting drops for 6 months after opening.

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© Bausch & Lomb Incorporated
®/™ trademarks of Bausch & Lomb Incorporated or its affiliates