**ENGLISH:**

STORE THERAPEARL PACK IN FREEZER  
OR AIRTIGHT PLASTIC BAG

**THERA°PEARL**  
A New Degree of Relief  
Doctor Designed Hot & Cold Therapy  
Reusable Hot & Cold Therapy

**Eye-ssential Mask**  
Conforms to your body to deliver soothing relief from:  
- Dry Eyes  
- Headaches / Sinuses  
- Puffy Eyes & More!

**Freeze & Microwave**  
Non- Toxic  
Reusable

[www.therapearl.com](http://www.therapearl.com), 0800-909-8180

Contains one (1) THERA°PEARL  
Eye-ssential Mask  
9 in x 2.75 in ( 22.9 cm x 7 cm )

**THERA°PEARL** combines the healing power of hot & cold therapy into one convenient pack. Enjoy extraordinary relief from your everyday aches & pains and feel the optimal therapeutic benefit as our Pearl Technology™ conforms to your body.

**INTENDED USE:**  
Reusable therapeutic hot & cold pack for use with common aches, pains, swelling and bruising. Packs are designed to retain a therapeutic temperature for approximately 20 minutes.

**COLD THERAPY ~** Freeze for at least 2 hours.

**HEAT THERAPY ~** Microwave for time indicated below and test pack temperature. Heat for additional 5 seconds as needed until desired temperature is reached. Suggested heating time for (1) one THERA°PEARL Eye-ssential Mask

700 Watts ~ 15 Seconds  
1000 Watts ~ 12 Seconds  
1250 Watts ~ 10 Seconds

**DIRECTIONS:**  
THERA°PEARL pack should be applied for 20 minutes, or as doctor recommended. Remove and wait for at least 20 minutes. Repeat as needed.

**CAUTION:** Always test pack temperature prior to application. Overheating or prolonged application may cause injury to person, including burns. For external use only. Do not use on infants. Do not puncture pack and/or ingest contents. Do not apply to sensitive skin, insensitive skin, or open wounds. Do not use for purposes other than intended. Do not fall asleep using pack. Use only under adult supervision. Heating beyond suggested time may damage or destroy THERA°PEARL pack. For extreme, prolonged, or chronic pain, contact a physician.

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