BAUSCH + LOMB

BAUSCH + LOMB Global Packaging + Graphic Design 1400 North Goodman Street, Rochester, NY 14609 USA

GLOBAL PACKAGING CONTACT: Amy Werther - e: amy.r.werther@bausch.com - t: 585.338.5202 - f: 585.338.8959

PRINTED BAR CODES MUST BE VERIFIED READABLE AND ACCURATE BY VENDOR TO THE HUMAN READABLE INDICATED ON ARTWORK.
THIS ARTWORK IS PROPERTY OF BAUSCH + LOMB. IT MAY NOT BE PRODUCED OR ALTERED WITHOUT EXPRESS PERMISSION FROM B+L.

DESCRIPTION: balafilcon A IFU / EMEA / ELC

PART No.: 8100401

SPEC No.: 5456

SPECIAL INSTRUCTIONS / PLACED IMAGES:

COATING PER SPECIFICATION

DIELINE DOES NOT PRINT

ARTWORK SET AT 100%

| | | | | | | | |
|-------------------|---------------------|--|--|--|--|--|--|
| BLACK 1 | PMS 320 2 | | | | | | |
|-------------------|---------------------|--|--|--|--|--|--|

**BAUSCH + LOMB
PUREVISION®**

EN

INSTRUCTIONS FOR USE

This leaflet is intended to inform users of Bausch + Lomb (balaflacon A) hydrophilic soft contact lenses how to wear and care for their lenses.

It applies to:

| PRODUCT NAME | MATERIAL | WATER CONTENT | PACKAGING SOLUTION |
|--|--------------|---------------|--|
| PureVision® (balaflacon A) Visibility Tinted Contact Lenses | balaflacon A | 36% by weight | Borate buffered saline |
| PureVision® Toric (balaflacon A) Visibility Tinted Contact Lenses | | | |
| PureVision® Multi-Focal (balaflacon A) Visibility Tinted Contact Lenses | | | |
| PureVision®2 (balaflacon A) Visibility Tinted Contact Lenses | balaflacon A | 36% by weight | Borate buffered saline with Poloxamine |
| PureVision®2 for Astigmatism (balaflacon A) Visibility Tinted Contact Lenses | | | |
| PureVision®2 for Presbyopia (balaflacon A) Visibility Tinted Contact Lenses | | | |

Description and wear application

These lenses are designed for daily wear with a replacement period as prescribed by the eye care professional. They may also be used as disposable lenses to be discarded after each removal or for extended wear for 1-30 days between removal, as recommended by the eye care professional. When reusing lenses they must be cleaned, rinsed and disinfected between removal and reinsertion.

The Bausch + Lomb (balaflacon A) Contact Lenses can also be used for therapeutic use in the management of corneal epithelial disorders. Always follow your eye care practitioner's recommended wearing schedule.

STERILE These lenses are supplied sterile and are intended by the manufacturer for reuse as prescribed by the eye care professional. They have been sterilized by steam.

If the packaging is damaged or opened DO NOT USE.

Insertion and removal instructions

Before handling lenses, always wash and dry hands thoroughly avoiding perfumed or medicated soaps. Pour the lens into the palm of your hand. Make sure the lens is the right way out.

Inspect your lens before inserting and do not insert if scratched or damaged.

To remove your lens, make sure your hands are clean and dry, use your finger to slide the lens down to the white part of the eye and remove it. Use a rewetting drop if the lens is difficult to remove.

Caring for your lenses

- Cleaning and disinfecting your lenses between uses is essential for healthy eyes
- Use only soft contact lens care products following the manufacturer's instructions
- Always use fresh, unexpired lens care solutions
- Do not use other solutions such as tap or bottled water or saliva to rinse, store or re-wet lenses
- Always keep lenses completely immersed in the recommended solution when not being worn
- Remember to clean your lens case whenever you clean your lenses
- Always discard lenses at the end of the wear schedule prescribed by your eye care practitioner
- Always wash and rinse hands before handling lenses
- Do not allow cosmetics, lotions, soaps, creams or perfumed substances to get in the eyes or on lenses
- Never use sharp objects or finger nails to remove lenses from their containers or your eyes

Warnings and precautions

Failure to follow your eye care practitioner's recommended wear schedule and cleaning regime can give rise to greater risk of serious eye infections.

Do not wear daily disposable lenses for longer than a day or while sleeping.

Do not wear contact lenses if you are suffering from:

- Allergy, inflammation, infection or redness in or around the eye
- Dry eyes (inadequate tear fluid)
- Poor health affecting the eyes, e.g. colds and flu
- Any systemic illness affecting the eyes

Contact your eye care practitioner before wearing contact lenses when:

- Practicing water sports
- In excessively dry or dusty environments
- Using eye medication

If you experience any of the following problems, remove your lenses immediately:

- Eyes stinging, burning, itching, irritation or other eye pain
- Reduced comfort compared with previous wearing
- Unusual secretions or excessive watering from the eye
- Redness of eyes
- Severe or persistent dry eyes
- Reduced or blurred vision
- Rainbows or halos around lights
- Sensitivity to light

If any of these symptoms continue after removal and insertion of a new lens, then remove the lens and contact your eye care practitioner. These symptoms can lead to more serious conditions if ignored.

Solution precautions

Do not use the Allergan Ultracare Disinfecting System or any of its components (Oxysept Ultracare Disinfecting Solution, Oxysept Neutralizing Tablets, and Ultrazyme Enzymatic Cleaner) to clean and disinfect Bausch + Lomb (balaflacon A) Contact Lenses because the lens dimensions will be altered.

Reference guide for symbols labels and cartons

Fee paid for waste management per 94/62/EC

Bausch & Lomb Incorporated
1400 North Goodman Street
Rochester NY 14609, USA

EC REP Bausch & Lomb Incorporated
106 London Road
Kingston-upon-Thames KT2 6TN, UK

Date of Issue: YYYY-MM-DD

CE 0050 Indicates the CE conformity marking and the notified body number

Temperature limitation

*™ denote trademarks of Bausch & Lomb Incorporated. © Bausch & Lomb Incorporated. Other product/brand names are trademarks of their respective owners.

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