BAUSCH + LOMB

BAUSCH + LOMB - Global Packaging + Graphic Design - 1400 North Goodman Street - Rochester, NY 14609 - USA
GLOBAL PACKAGING CONTACT: Bruce A Ballard - e: bruce_a_ballard@bausch.com - t: 585.338.5839 - f: 585.338.8959

PRINTED BAR CODES MUST BE VERIFIED READABLE AND ACCURATE BY VENDOR TO THE HUMAN READABLE INDICATED ON ARTWORK. THIS ARTWORK IS PROPERTY OF BAUSCH + LOMB. IT MAY NOT BE PRODUCED OR ALTERED WITHOUT EXPRESS PERMISSION FROM B+L.

DESCRIPTION: ReNu MPS EN/SP/PO Insert Iberia MLN
PART No.: I54084651 **SPEC No.:** 55147
REVISION: 02 **DIMENSIONS:** 120 x 170

DCO No.: 291A
EUROLASER Ref.:

APPROVAL SIGNATURES

SPECIAL INSTRUCTIONS:

COATING PER SPECIFICATION

DIELINE DOES NOT PRINT

ARTWORK SET AT 100%

PLEASE READ CAREFULLY AND KEEP THIS PACKAGE INSERT FOR FUTURE REFERENCE.

BAUSCH + LOMB

ReNu MPS®
Multi-Purpose Solution

DIRECTIONS: To clean and disinfect, complete these simple steps.

STEP 1: Place 2 drops of ReNu MPS Multi-Purpose Solution on each side of the lens surface and gently rub for 20 seconds.

STEP 2: Thoroughly rinse each side of the lens with ReNu MPS Multi-Purpose Solution.

STEP 3: Place cleaned contact lens in the lens case and fill with fresh ReNu MPS Multi-Purpose Solution. Soak at least 4 hours. Remember to always use fresh solution.

Your lenses are now ready to wear. No saline rinse necessary. If any debris remains on contacts, rinse with ReNu MPS Multi-Purpose Solution prior to insertion.

Always follow your eye care practitioner's instructions. Based upon your individual wear chemistry and lens-wearing schedule, your eye care practitioner may recommend additional products or procedures.

WEEKLY CLEANING: Protein removal with an enzymatic cleaner may be necessary for some patients. Please consult your eye care practitioner.
 Note: Proper lens care requires removal of protein deposits from your lenses.

STORAGE: You may store your lenses in the unopened lens case until ready to wear, up to a maximum of 30 days.

CONTENTS: A sterile, isotonic solution that contains boric acid, edetate disodium, polyoxamine 1%, sodium borate and sodium chloride; preserved with DYMED® (polyaminopropyl biguanida) 0.00005%.

ACTION: Cleans, loosens and removes accumulations of film, deposits and debris from soft contact lenses. Destroys harmful micro-organisms on the surface of the lens. Rinses, stores and rewets lenses before insertion.

INDICATIONS (Uses): BAUSCH + LOMB ReNu MPS Multi-Purpose Solution is indicated for use in the daily daily cleaning, rinsing, and storage of daily and extended wear soft contact lenses and chemical (not heat) disinfection, as recommended by your eye care practitioner.

CONTRAINDICATIONS (Reasons not to use): If you are allergic to any ingredient in this product, do not use.

WARNINGS: PROBLEMS WITH CONTACT LENSES AND LENS CARE PRODUCTS COULD RESULT IN CORNEAL INFECTION AND/OR ULCER AND LEAD TO LOSS OF VISION. It is essential that you follow all labeling instructions for proper use of lenses and lens care products, including the lens case.

Eye problems, including corneal ulcers, can develop rapidly and lead to loss of vision. Daily wear lenses are not indicated for overnight wear and should not be worn while sleeping. Clinical studies have shown the risk of serious adverse reactions is increased when these lenses are worn overnight. Extended wear lenses should be regularly

removed for cleaning and disinfection or for disposal and replacement on the schedule prescribed by your eye care practitioner. Clinical studies have shown that there is an increased incidence of serious adverse reactions in extended wear contact lens users as compared to daily wear contact lens users. Studies have also shown that the risk of serious adverse reactions increases the longer extended wear lenses are worn before removal for cleaning and disinfection or for disposal and replacement. Studies have also shown that smokers have a higher incidence of adverse reactions. If you experience eye discomfort, excessive tearing, vision changes, redness of the eye, immediately remove your lenses and promptly contact your eye care practitioner. It is recommended that contact lens wearers see their eye care practitioner twice each year, or if directed, more frequently.

You should fill your lens case with fresh solution every time you store your lenses, and never "top-off" or re-use solution. You should discard your solution immediately after your lenses have been removed from the lens case. You should not expose or store your lenses in or rinse your lens case with any water, such as tap, bottled or distilled, or with any non-sterile solution.

Empty, then clean and rinse your lens case with ReNu MPS Multi-Purpose Solution. Air dry your lens case each time you remove your lenses. In order to permit excess solution to drain, you can flip over your lens case while air-drying. Replace your lens case monthly.

Warning: Failure to discard solution from lens case after each use or use of water to care for your lenses may lead to contamination resulting in eye injury and potential loss of vision. See accompanying instructions for additional important safety information.

- PRECAUTIONS:**
- Always follow the product directions for use. Failure to follow product directions may lead to vision loss.
 - Visit your eye care practitioner regularly.
 - Always wash and dry hands before handling lenses.
 - Do not use tap water, bottled water or saliva with lenses or lens case.
 - Only use fresh solution to clean and disinfect contact lens.
 - Always discard any remaining solution in your lens case after each disinfection cycle.
 - Saline or rewetting drops will not disinfect your lenses.
 - Always replace your solutions and lenses as directed.
 - Always replace your lens case monthly.
 - To avoid contamination, do not touch tip of container to any surface.
 - Replace cap after use.
 - Do not use any eye medication in conjunction with this solution unless under medical supervision.
 - Not for use with heat (thermal) disinfection.
 - Keep the bottle tightly closed when not in use.
 - Use before expiration date marked on the carton and bottle.
 - Discard remaining solution within 90 days after opening.
 - Keep out of reach of children
 - Store at room temperature

IMPORTANT: If irritation or excessive tearing occurs, persists or increases, or if vision is impaired, discontinue use and promptly consult your eye care practitioner.

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©/™ are trademarks of Bausch & Lomb Incorporated or its affiliates.